



**VetPen®**

WITH YOU AND  
YOUR PET  
**FOR LIFE**

Your pet has been diagnosed with diabetes.\* Pet diabetes is a common condition which needs you to work closely with your veterinary practice to successfully manage.

Insulin can be administered to your pet via two methods; either by syringe or, much like in human diabetes, with an insulin pen.





An insulin pen can help to make insulin injections much easier for you and may be more comfortable for your pet.<sup>1</sup>

**If your pet's diabetes is not regulated and you are struggling with using insulin syringes, talk to your vet about other delivery options.**

**It is not recommended to switch insulin delivery devices if your pet's diabetes is stabilised.**



VetPen® is exclusively designed and calibrated for pets to deliver comfort and accurate insulin dosing.<sup>2</sup>

## The advantages of using VetPen:

### 1 Could help to minimise pet discomfort and reduce injection pain.


VetPen needles are high-tech, triple sharpened and coated in silicone which can allow for a smoother, pain-free injection. This may also reduce the scarring/thickening of skin at injection sites that may occur over the long term.



### 2 Dosing is as easy as turning a dial.

97% of pet owners reported VetPen is easy to use.<sup>1</sup>



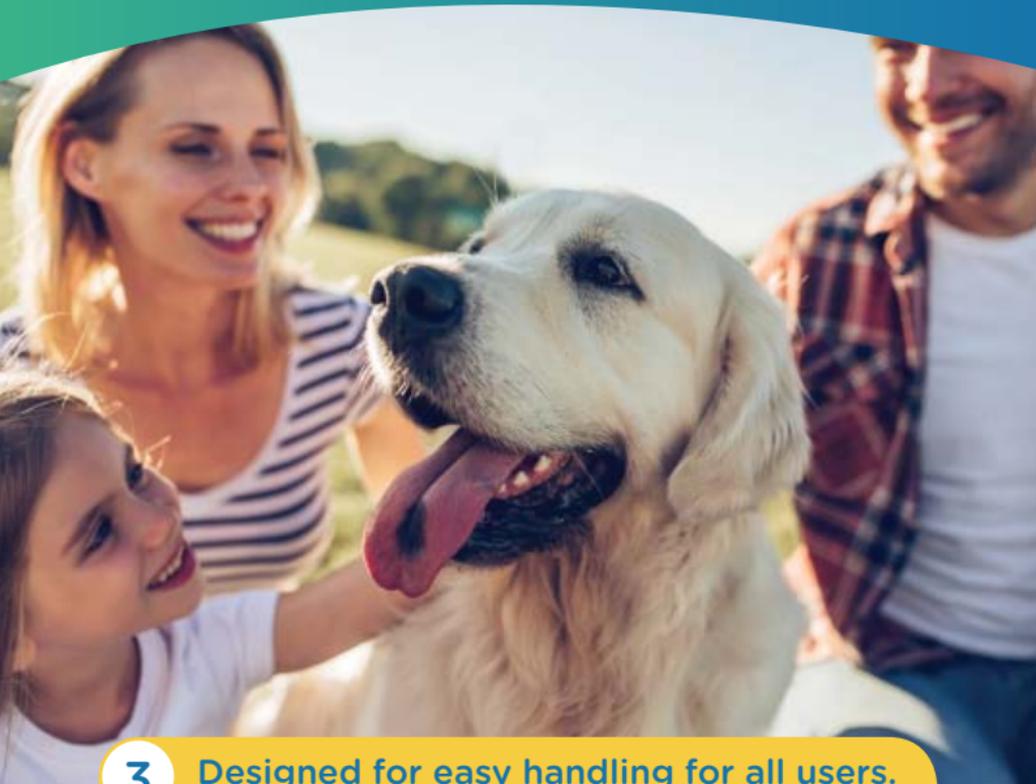

**3** Designed for easy handling for all users.

**4** Easy to train other pet-carers.

When you need to rely on others to administer insulin to your pet, for example when you leave your pet with a kennel or cattery, or when a friend or family member needs to perform an injection, instructing others to use VetPen is easy.



## 5 Especially helpful if you have poor eyesight or unsteady hands.

VetPen includes adaptors designed to fit over the release button and dose selector to make handling easier. Also, when a dose is dialled up on VetPen it can be audibly heard as “click” sounds which enables the user to gauge how much insulin is being set to be administered.



## 6 Improved user safety.

Injuries from needles can be a hazard with diabetes management, and needle stick injuries can be quite common. A needle remover is included with VetPen to ensure safe removal of needles. Needles have double needle caps for added safety.



## 7 Improving dose accuracy and consistency.<sup>2</sup>

In clinical trials, VetPen has been proven to be more accurate in dosing than syringes.



**8** Portable and easy to take on the go.

VetPen comes with a handy carry case.



**VETPEN MAKES INSULIN  
INJECTION SIMPLE, EASY  
AND MORE ACCURATE.<sup>1,2</sup>**





**For further information  
speak to your veterinary  
practice.**

\*Your vet has prescribed Caninsulin® for your diabetic pet.

References:

1. Lust E., van Esch K., Slingerland L.I., Kooistra H.S., Horspool L.J.I. (2017). Survey of Veterinary Professional and Pet Owner VetPen User Experience. Presented at the 42nd WSAVA and FECAVA 23rd Eurocongress; Copenhagen, Denmark.
2. Burgaud, S., Riant, S. & Piau, N. (2012). Comparative laboratory evolution of dose delivery using a veterinary insulin pen. WSAVA Poster.

This information is provided by MSD Animal Health, the manufacturer of Caninsulin®.

GB/SMA/0619/0149

